Advantages of ello

On the basis of the settings menu

ello is a rollator which offers more security and mobility already with its basic functions. But each user has another view with regards to speed or another need for stability. Additional functions of ello can be activated in the settings menu which make the rollator an individual product. This means that user requirements can be met completely.

Individual speed

ello does never run faster than the maximum speed and stays always close to you. A safe companion.

You can determine a maximum speed and a minimum speed. This means that you can tailor the speed range to your needs. With a wide range you can cover low speeds as well as high speeds. With a small range you determine yourself which speed is best for you and ello remains in this speed range in any case.

Lane keeping function ello supports your directional stability. More security also one-handed.

With the electric drive of ello you can adjust how much ello supports you when steering or moving straight ahead. With the lane keeping function you can cope with uneven surfaces and steep kerbs. This allows you to easily steer ello with one hand. ello supports you when moving straight ahead and with that helps to compensate an unsteady gait. You regain security when walking.

Automatic braking function ello avoids falls - event without drive activation.

You can push ello also without electric drive - you can use ello as a conventional rollator. Also without electric drive, ello can avoid falls. ello registers the speed and compares it with

the adjusted speed (+/- button, blue lights). ello recognizes when you are falling and activates the brake. You can moderate falls or avoid them completely. In any case, ello does not roll away even if you park ello on a gradient.





3/F, Unit A, Kader Building, 22 Kai Cheung Road, Kowloon Bay, HK Phone +852 2511 2118 | sales@dpx.hk | www.deltapyramax.com



the electric rollator for more comfort and mobility



"ello is the new premium class among the rollators"

Developed and produced in Germany

ello is the electric rollator that offers **more user convenience** – supports mobility and quality of life via simple and intuitive operability.

With light, horn and SOS function* the user is noticed better in daily life and this **improves his security and independence**. The walking speed of ello can be adjusted individually. As a result, the user saves energy for braking and falls can be avoided.

The **additional climbing aid** (e.g. for kerbs) allows the user to **overcome obstacles** easily. ello can be operated also with one hand only and frees up the user in his movement coordination. In addition, the user is protected by the **integrated emergency system*** in all every life situation.

With the active sale of our ello at your premises you have the possibility to **represent yourself innovatively** and up-to-date.

E-mobility is THE future topic and you enlarge and perfect your rollator product range with ello. With this enlargement of your product range you have the possibility to offer your existing clients an **innovative and modern product.**

In addition, you can extend your new business and achieve a competitive advantage. And the great thing is that you are supported by us with active product communication.

Always one step ahead

Ready to start in an instant

1. Insert battery

The round form of the battery cannot be inserted incorrectly. The battery snaps by turning into the holder.

2. Switch on ello

The keypad has a clear format, with large symbols for easy handling.

3. Choose speed

Adjust the desired speed via the plus and minus buttons.

4. Apply thumb & start

For your safety, ello only starts when you put your thumb on the sensor. In case of need, ello stops immediately as soon as you remove the thumb.

ello offers many unique advantages – for more comfort and unlimited mobility:



Help at the push of a button*:

In case of an emergency ello sends an emergency call with the GPS data to the stored contacts. The SOS function offers more security and can save lives.



Inconspicuous technology:

The motor is in the rear wheels and is so powerful that ello runs uphill and brakes downhill. In addition, obstacles can be overcome more easily.



Mobile without limits:

The battery has a range of 10 km and can be charged easily overnight (6 hours). In case the battery is empty, ello can be operated as a conventional rollator.

* only in Europe * only in Europe